



LIFESENSE®

## TENS REPLACEMENT PADS

Use with Medicare TENS  
Therapy Device



### FACT BOX

Product Code:

MD1995

Product Category:

Diagnostics

#### Contents:

- 1 x TENS Device
- 2 x Electrode Wires
- 4 x Electrode Pads
- 4 x AAA Batteries
- 1 x Storage Bag

#### Warning:

Do not apply the pads on both sides of the chest cavity simultaneously (lateral or front and back), or across your chest as the introduction of electrical current may cause rhythm disturbances which could be lethal.

Do not apply the pads near the heart. Do not apply the pads on the calves of both legs at the same time.

If you have had medical or physical treatment for your pain, consult with your physician before using this device.

If your pain does not improve, becomes more than mild, or continues for more than five days, stop using the device and consult with your physician.

The Medicare LifeSense® TENS Therapy Device employs Transcutaneous Electrical Nerve Stimulation (TENS) to provide drug-free pain relief by utilising a gentle electrical current.

This innovative technology is designed to alleviate pain and muscle spasms associated with various conditions such as arthritis, menstrual discomfort, endometriosis-induced pelvic pain, knee pain, sports injuries, and is occasionally utilised for pain relief during childbirth.

#### FEATURES:

- Pain relief and treatment
- Reusable up to 30 times
- For use with MD1990 Medicare TENS Therapy Device

#### DIRECTIONS:

1. Tear open the plastic bag containing the electrode pads.
2. Connect the electrode wires to the electrode pads.
3. Connect the electrode wires to the TENS therapy device. (Remove the pads protective film).
4. Attach the pads to the required locations for therapy. Consult the TENS therapy device manual for more information.

#### CONTRAINDICATIONS AND PRECAUTIONS:

Patients with skin diseases, malignant tumours, allergies to electrode pads, and implanted cardiac pacemaker are forbidden to use the device. Consult your doctor for other disabled situations. Further information can be found in the user manual.



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid