



V3 Wearable Nebuliser

Hands-free & easy-to-use



FACT BOX

Product Code:
MD2008

Product Category:
Diagnostics

Caution:

Clean and disinfect the medicine cup and inhalation mask, after every use.

This device is designed for personal use. It is not appropriate for more than one user.

Do not wash the main unit with water. Do not immerse them in water. The device may experience electricity leakage or you may suffer electric shock. The device can become damaged and may no longer be able to be used.

Contents:



Medicine Cup




Power Unit & Connection Cable



Adult Mask



Child Mask

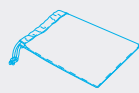

Storage Pouch

Phone Connectors:



Lightening



USB-C

The Medicare® V3 Nebuliser is designed with your convenience in mind. Enjoy hands-free operation for ultimate ease and mobility, allowing you to go about your day without interruption.

Powered by your phone, battery pack or USB cable, this nebuliser ensures flexibility and adaptability to your lifestyle.

FEATURES:

- Versatile power options
- Quiet operation
- Tailored to meet your respiratory needs
- Comfortable and secure inhalation mask

DIRECTIONS:

1. Fill the medicine cup with the medication prescribed.
2. Connect the power supply to the medicine cup.
3. Install the inhalation mask to the medicine cup.
4. Place the mask over the face.
5. Start inhalation and then short press the power button when finished.

CONTRAINDICATIONS AND PRECAUTIONS:

This device is a professional atomisation treatment equipment. Children and adults who are unable to take care of themselves should ensure that a guardian is present when using it.



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid