



Travel & Motion Sickness Band

Stops Travel Sickness before it starts



FACT BOX

Product Code:
MD4000198

Product Category:
Home Health

Contents:
1 pair of Sickness Bracelet

Danger:

The band provides symptomatic relief, it will not cure or remove the cause of the nausea.

Nausea and vomiting can be associated with serious conditions. If your symptoms persist, consult a physician at once.

If skin irritation occurs beneath the band, or if pain or swelling occurs, remove the bands immediately and consult a physician if these symptoms persist.

Store out of reach of children.

The button is a choking hazard if removed.

Individuals with known or suspected circulation problems should consult a physician prior to use.



Medicare® Travel & Motion Sickness Band is intended for the relief of nausea. Nausea may be experienced due to travel/motion sickness, anaesthesia or chemotherapy. It operates by exerting pressure on the Nei-Kuan (acupressure) point on each wrist.

FEATURES:

- One size fits all
- No Drugs and Non Drowsy

DIRECTIONS:

1. To find your Nei-Kuan point, place your middle three fingers on the inside of your wrist with the edge of the third finger just below the first wrist crease.
2. The Nei-Kuan point is located underneath your first finger in between the two wrist (flexor) tendons (A).
3. Position the button facing inward over the Nei-Kuan point (B). Repeat the process for the other wrist.

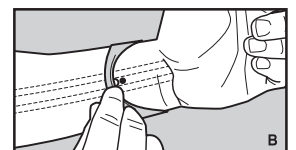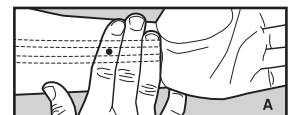

CONTRAINDICATIONS AND PRECAUTIONS:

One band must be worn on each wrist to be effective.

Store out of reach of children, the button is a choking hazard if removed.

The band provides symptomatic relief, it will not cure or remove the cause of the nausea.