



Adhesive Remover Spray

Gentle & Skin-Friendly



FACT BOX

Product Code:
MD503

Product Category:
First Aid

Size:
35ml

Composition:
Disiloxane, Butane, Propane,
Isobutane, Cyclopentasiloxane.

Intended User:
Medicare Adhesive Remover Spray can be used for the entire population, for young and newborn babies, for diabetics. It is designed for home and professional use.

Shelf Life:
3 years.
Up to the date indicated on the bottom of the packaging.

Precautions:
In case of chronic wounds it is recommended to be used by related person only to maintain hygiene standards.
May cause an allergic skin reaction in hypersensitive persons.
Extremely flammable aerosol.

The Medicare Adhesive Remover Spray offers gentle removal of medical adhesives like patches and bandages. Its residue-free formula minimises skin irritation, ensuring hassle-free and comfortable adhesive removal.

FEATURES:

- It is used for easy removal of sticking plasters, transdermal patches, adhesive bandages and other medical aids attached to the body surface.
- Evaporates completely.
- Does not affect the healing process and it can not be absorbed into the wound.
- Can be used on sensitive, very dry, cracked or damaged skin.
- It leaves the treated area clean and ready for applying a new dressing.

DIRECTIONS:

1. The skin around the patch should be clean and dry before using the spray.
2. Spray from a distance of 10-15 cm away from the skin.
3. Wait for 10-15 seconds and remove the patch quickly.

CONTRAINDICATIONS:

Do not use in case of allergic reaction to any of the ingredients. For use in pregnant and breastfeeding women consult the physician.



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid