



# Foam PU Earplugs

Protects from harmful  
or irritating sound



## FACT BOX

**Product Code:**  
MD70

**Product Category:**  
Home Health

**Contents:**  
2 pairs of Foam PU Earplugs

**Danger:**  
This product may be adversely  
affected by certain chemicals.

Keep out from water.

Earplugs should be worn at all times  
in noisy surroundings and be fitted  
prior to entering the noisy area.

Sudden or fast removal of the  
earplugs out of the ear canal may  
damage the ear drum.

If you have a skin irritation in your  
ear canal, please contact your  
physician before use.

Store out of reach of children. There  
is a risk of choking if the plug is  
stuck in the windpipe.

Failure to comply with the above  
instructions for use may impair the  
efficiency of the earplugs.

For further information contact  
Fleming Medical.

Medicare® Foam PU Earplugs are designed to prevent disturbances in the hearing conduct caused by excessive noise. They are gentle, comfortable and adapt perfectly to the shape of the ear, offering excellent protection against external agents such as dust, wind and noise.

## FEATURES:

- One size fits all
- 2 pairs included
- Protects from irritating or harmful sound

## DIRECTIONS:

1. Roll and compress the earplug between the fingers into a smooth, crease-free cylinder.
2. To ensure a proper fit reach hand over head and gently pull ear upward and outwards.
3. Insert compressed, rounded end of the earplug well into the ear canal. The pointed end should go in first.
4. Hold for a few seconds while plug expands.

## CONTRAINDICATIONS AND PRECAUTIONS

Keep out of the reach of children.

Do not push the plug deep into the ear canal.

Sudden removal may damage eardrum.

Keep out from water.



@Medicarefirstaid



@Medicarefirstaid



@Medicare1staid