



Foam PVC Earplugs

Protects from harmful or irritating sound



FACT BOX

Product Code:
MD72

Product Category:
Home Health

Contents:
2 pairs of Foam PVC Earplugs

Danger:
The earplugs may be worn constantly in noisy environments. Foam earplugs are not designed to protect against water. Do not push the plug deep into the ear canal. Always remove the earplugs slowly from the ear. Keep away from heat and direct light. Keep out of the reach of children. Consult doctor before using if you have or have had ear problems (infections or perforated eardrum). This product may be adversely affected by certain chemical substances. Failure to comply with the listed instructions for use may impair the efficiency of the earplugs. For further information contact Fleming Medical.

Medicare® Foam PVC Earplugs are designed to prevent disturbances in the hearing conduct caused by excessive noise. They are gentle, comfortable and adapt perfectly to the shape of the ear, offering excellent protection against external agents such as dust, wind and noise.

FEATURES:

- One size fits all
- 2 pairs included
- Protects against water and irritating sounds

DIRECTIONS:

1. Roll and compress the earplug between the fingers into a smooth, crease-free cylinder.
2. Whilst keeping the earplug compressed, gently insert it into the ear.
3. Hold the earplug in the ear for 15-20 seconds until plug fully expands.

CONTRAINDICATIONS AND PRECAUTIONS

Keep out of the reach of children.
Do not push the plug deep into the ear canal.
Sudden removal may damage eardrum.
Keep away from direct light and heat.