PHYSIO LOGIX®

Professional Sports Tape

Medium to Firm Compression & Support for Muscles & Joints



FACT BOX

Product Code:

PX401-PX403 - White PX405 - Beige PX408, PX411 - Red PX412, PX415 - Blue PX417-PX419 - Black

Product Category:

Sports Therapy

1 x 2.5cm x 5m Roll

Contents:

(PX412, PX418) 1 x 5cm x 5m Roll (PX401, PX405, PX417) 1 x 7.5cm x 5m Roll (PX402, PX418)

1 x 10cm x 5m Roll (PX403, PX411, PX415, PX419)

Precautions:

If any irritation occurs discontinue use. Do not try to treat serious injuries at home. Do not apply tape too tightly. Remove tape if numbness or tingling occurs, and re-apply less tightly. If in doubt about an injury, seek medical advice.

Fibre Content:

90% Cotton, 10% Spandex.

The Physiologix Professional Sports Tapes are a strapping tape that are self-adhesive and provide firm support for sports, sprains and strains.

FEATURES:

- Self-Adhesive: Sticks to itself and skin with medium adhesion
- Elasticated: Stretchy material offers varied compression levels
- Flexible: Provides comfort and freedom of movement

DIRECTIONS:

- 1. Use as a compression, support, injury prevention or relieving bandage.
- 2. Unwind and wrap around affected area.
- 3. Ideal for ankles, knees and elbows.

REST

Avoid weight bearing activity on the injured area in the first 24 to 48 hours of the injury occurring.



ICE

Ice the injured area with a cold compress for 15 minutes every 2 to 3 hours.



COMPRESS

Compress the injured area with a strapping tape to help limit swelling and movement.



ELEVATE

Elevate the injury, resting above heart level and keep supported. This will further help reduce any swelling.







